


Transvaginal Mid Urethral Sling Procedure


Ease of Use. Reliable Performance.

1

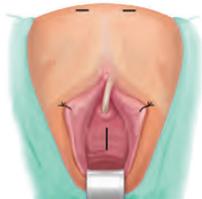


Blunt or sharp paraurethral dissection is performed to allow the Transvaginal Introducer to pass through the Space of Retzius. Two stab incisions are created over the pubic symphysis approximately 1-2 cm lateral of midline.

2

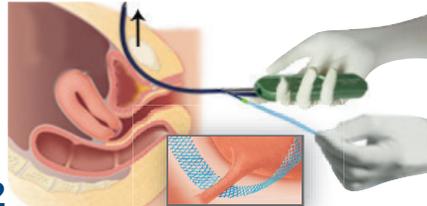


Locate the aperture on the dilator tube, and place one dilator tube assembly over the Transvaginal 3.2 mm Introducer. Pass the Transvaginal Introducer and dilator tube assembly through the vaginal incision up through the retropubic space adjacent to the posterior surface of the pubic symphysis and through the suprapubic incision. Maintain the introducer position close to the pubic bone to avoid any adverse events. Make sure the mesh is lying flat under the urethra and is not twisted.

3

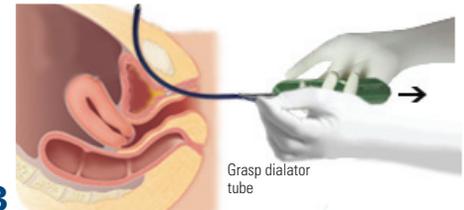


While Desara TV is in place, grasp the dilator tube at the junction with the sleeve where colored green, and slowly withdraw the rigid introducer out of the dilator tube through the vaginal incision. Alternatively, you can grasp the dilator tube tip at the suprapubic exit point while withdrawing the introducer.

4

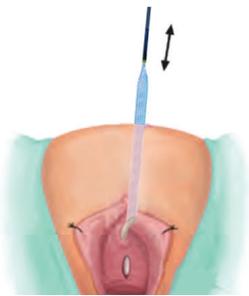


Once the introducer is removed, place a clamp on the dilator tube tip to hold it in place prior to placing the opposite side.

5

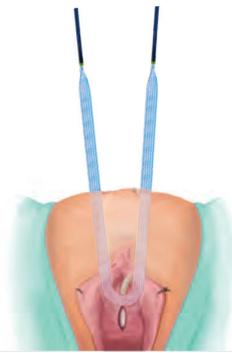

Repeat steps 3 - 7 on the contralateral side. Cystoscopy should be performed to rule out any bladder perforations. Place the sling in a tension free manner under the mid urethra.

6

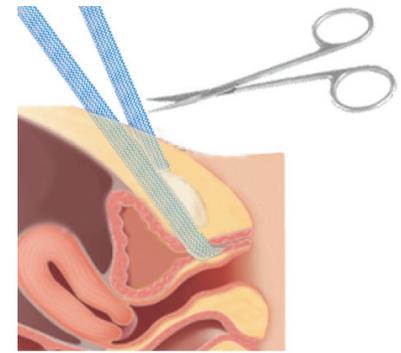

Cut off the ends of the mesh assembly medial to the tip and dilator tube and pull off the sleeve. Trim the excess mesh that extends beyond the suprapubic incisions so the mesh lies below the skin. Incisions are closed according to usual methods.

ORDER NUMBER	DESCRIPTION	QUANTITY
CAL - DS01BTV	Desara Blue TV	Box of 5
CAL - DS01TV	Desara TV	Box of 5

For product evaluation, call your Caldera Sales Representative today at (866) 4-CALDERA

*Data on file at Caldera Medical